

DRILLING PROTOCOL FOR CONTACT

| PF \varnothing 3.5 endosseous \varnothing 2.7 | PF \varnothing 4.0 endosseous \varnothing 3.5 | PF \varnothing 4.5 endosseous \varnothing 3.5 | PF \varnothing 5.0 endosseous \varnothing 4.2 | PF \varnothing 6.0 endosseous \varnothing 5.0 | Rotational speed rpm |
|--|--|--|--|--|----------------------|
| 2.0 Pilot drill | 2.0 Pilot drill | 2.0 Pilot drill | 2.0 Pilot drill | 2.0 Pilot drill | 800 |
| ↓ | 2.8 VECTOdrill | 2.8 VECTOdrill | 2.8 VECTOdrill | 2.8 VECTOdrill | 600 |
| | ↓ | ↓ | 3.5 VECTOdrill | 3.5 VECTOdrill | 500 |
| 4.3 VECTOdrill | | | 400 | | |
| 2.7/3.5 CONTACT Profile drill | 3.5/4.0 CONTACT Profile drill | 3.5/4.5 CONTACT Profile drill | 4.2/5.0 CONTACT Profile drill | 5.0/6.0 CONTACT Profile drill | 250-300 |

