

PROTOCOLLO CHIRURGICO PER CONTACT

| PF Ø 3.5 Ø endosseo 2.7 | PF Ø 4.0 Ø endosseo 3.5 | PF Ø 4.5 Ø endosseo 3.5 | PF Ø 5.0 Ø endosseo 4.2 | PF Ø 6.0 Ø endosseo 5.0 | Numero di giri g/min |
|-----------------------------------|-----------------------------------|-----------------------------------|-----------------------------------|-----------------------------------|----------------------|
| 2.0 Fresa pilota | 2.0 Fresa pilota | 2.0 Fresa pilota | 2.0 Fresa pilota | 2.0 Fresa pilota | 800 |
| ↓ | 2.8 VECTOdrill | 2.8 VECTOdrill | 2.8 VECTOdrill | 2.8 VECTOdrill | 600 |
| | ↓ | ↓ | ↓ | 3.5 VECTOdrill | 500 |
| 4.3 VECTOdrill | | | | 400 | |
| 2.7/3.5 Fresa per profilo CONTACT | 3.5/4.0 Fresa per profilo CONTACT | 3.5/4.5 Fresa per profilo CONTACT | 4.2/5.0 Fresa per profilo CONTACT | 5.0/6.0 Fresa per profilo CONTACT | 250-300 |

