

PROTOCOLO DE FRESADO **CONTACT**

PF Ø 3.5 Ø endoóseo 2.7	PF Ø 4.0 Ø endoóseo 3.5	PF Ø 4.5 Ø endoóseo 3.5	PF Ø 5.0 Ø endoóseo 4.2	PF Ø 6.0 Ø endoóseo 5.0	Velocidad de rotación rpm
2.0 Fresa piloto	2.0 Fresa piloto	2.0 Fresa piloto	2.0 Fresa piloto	2.0 Fresa piloto	800
↓	2.8 VECTOdrill	2.8 VECTOdrill	2.8 VECTOdrill	2.8 VECTOdrill	600
	↓	↓	↓	3.5 VECTOdrill	500
4.3 VECTOdrill				400	
2.7/3.5 Fresa de perfil CONTACT	3.5/4.0 Fresa de perfil CONTACT	3.5/4.5 Fresa de perfil CONTACT	4.2/5.0 Fresa de perfil CONTACT	5.0/6.0 Fresa de perfil CONTACT	250-300

